

## #8800 MULTI-ACUITY PROJECTOR SLIDE INSTRUCTIONS

The Stereo Optical Multi-Acuity Test Slide has been designed for use with Marco, A.O. (Reichert), Woodlyn and similar projectors.

### CONVERT FOR USE IN A 10 FOOT LANE

Although initially configured for use in a 20 foot lane, the 8800 test slide can be easily converted for a 10 foot lane.

1. The glass slide is held in the frame by a spring in the bottom channel.
2. Remove the glass slide from the frame as follows:  
Grasp slide with one hand and frame with the other.  
Slide glass down within frame.  
Tilt top of slide out and remove.
3. Rotate glass slide on a vertical axis.
4. Return slide to frame by reversing process in step 2.

### CLEANING INSTRUCTIONS

Do not immerse slide in any liquid or spray liquid directly on slide, use a soft damp cloth to clean.



8400  
Acuity  
Astigmatism



8500  
Adult  
Acuity



8600  
Child  
Acuity



8700  
Family  
Practice



8800  
Multi-Acuity



8900  
Acuity  
Vergence

## #8800 MULTI-ACUITY PROJECTOR SLIDE ANSWER KEY

### LETTER ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/400	E
20/200	E
20/100	H B
20/80	C A V 8
20/70	D F G 7
20/60	D A O 6
20/50	E G N U 5
20/40	F Z B D 4
20/30	O F L C 3
20/25	A P S O 2 5
20/20	E V O T Z 2
20/20	T E G A D M
20/20	H X F Z H G
20/20	T Z V E C L
20/15	O H P N T Z

### TUMBLING "E" ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/200	RIGHT
20/100	BOTTOM RIGHT
20/80	LEFT TOP RIGHT BOTTOM
20/70	RIGHT BOTTOM LEFT TOP
20/60	BOTTOM TOP LEFT TOP RIGHT
20/50	TOP BOTTOM RIGHT TOP BOTTOM
20/40	LEFT TOP BOTTOM RIGHT LEFT
20/30	RIGHT LEFT TOP BOTTOM RIGHT
20/25	BOTTOM TOP LEFT TOP LEFT

### ALLEN PRESCHOOL ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/200	CAKE
20/100	HAND PHONE
20/80	BIRD HORSE
20/60	CAKE CAR HAND
20/40	PHONE BIRD HORSE
20/30	CAKE CAR HAND PHONE BIRD



**STEREO OPTICAL CO., INC.**

8623 W. Bryn Mawr Ave., Suite 502

Chicago, IL 60631

800-344-9500

Fax 773-867-0388

[www.stereooptical.com](http://www.stereooptical.com)

[sales@stereooptical.com](mailto:sales@stereooptical.com)